TECHNISCHE INFORMATIONEN

DE

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EN

1. INTENDED USE

The 101-VAL-CS One-way Valve for Confidence Socket may only be used for transtibial prosthetic fittings with the Confidence Socket.

2. PACKAGING CONTENT

The 101-VAL-CS One-way Valve for Confidence Socket consists of:

- Valve body with valve
- 4 x Countersunk Bolt DIN 7991 M6x16 12.9
- ca. 1,2 rmt 901-MT Molding Textile.

3. ASSEMBLY

1. Ensure that the valve is screwed firmly into the valve body with a 4mm Allen wrench.
2. Push the valve body into the hole in the silicone pad at the inside of the Confidence Socket.
3. The valve should sit flush with the silicone distal pad.

4. MAINTENANCE

1. Unscrew the valve out of the valve body with a 4mm Allen wrench.
2. Take out the seal of the valve.
3. Clean the valve with compressed air, assemble it again and fix it in the socket.

5. RISKS

If the given instructions are followed, no particular risk is expected when using the 101-VAL-CS One-way Valve for Confidence Socket.

6. WARRANTY

Amparo GmbH offers a warranty of 12 months for orthopedic workshops only. This warranty applies to faults in material and workmanship.

7. DECLARATION OF CONFORMITY

The product complies with the requirements of Directive MDR 2017/745. Based on the classification criteria for medical devices according to Annex IX of the Directive, the product has been classified as Class I. The Declaration of Conformity has therefore been drawn up by Amparo GmbH under its sole responsibility in accordance with Annex VII of the Directive.

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Az Amparo GmbH 12 hónapos garanciát csak szakcégek számára vállal. A garancia gyártási és anyaghibákra vonatkozik.