TECHNISCHE INFORMATIONEN

DE

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EN

1. INTENDED USE

The 101-VAL-TS One-way Valve for Trust Socket is designed exclusively for use with transtibial prostheses. It pushes excess air out of the socket during the stance phase. It closes during the swing phase. Disturbing friction and shear forces are thus reduced.

2. COMPONENTS

The 101-VAL-TS One-way Valve for Trust Socket consists of:

- Valve body with valve
- O-ring seal for valve body
- Counter nut M10 with O-ring
- compensation unit and
- dust filter.

3. ASSEMBLY

1. Drill a 10 mm diameter hole from the outside into the socket in the desired valve position.
2. Gently heat the inside surface of the socket near the hole for approx. 8 seconds.
3. Remove the O-ring from the valve body.
4. Insert valve body with valve from inside.
5. Tighten lock nut with the smooth side facing the socket from the outside on the valve body.
6. Tighten with 4 mm Allen key until the valve body is flush with the inner socket.
7. After the socket has cooled, loosen the lock nut and unscrew the valve body with valve.
8. Place the O-ring on the valve body and reinsert it into the socket.
9. In the case of strongly conical stumps, the compensation unit can also be placed on the valve body with valve between the socket and the lock nut.
10. For final assembly, tighten the lock nut with the O-ring side facing the socket on the valve body with valve.
11. Tighten the valve body with valve using a 4 mm Allen key.
12. insert the dust filter into the valve socket.

4. MAINTENANCE

1. Remove the dust filter if the valve is dirty.
2. Unscrew the valve from the valve body using a 4 mm Allen key.
3. Remove the gasket from the valve.
4. Clean the valve with compressed air, mount it and screw it into the socket.
5. Insert the dust filter.

5. RISKS

If the given instructions are followed, no particular risk is expected when using the 101-VAL-TS One-way Valve for Trust Socket.

6 WARRANTY

The 101-VAL-TS One-way Valve for Trust Socket has been tested according to ISO recommendations. A warranty of 6 months is only given by Amparo GmbH to specialist companies. The warranty refers to manufacturing and material defects.

7. DECLARATION OF CONFORMITY

The product complies with the requirements of Directive 93/42/EEC for medical devices. Based on the classification criteria for medical devices according to Annex IX of the Directive, the product has been classified as Class I. The Declaration of Conformity has therefore been drawn up by Amparo GmbH under its sole responsibility in accordance with Annex VII of the Directive.