

## INFO for Amputees

---

# CONFIDENCE SOCKET

### Your socket is extremely heat-sensitive.

It should not be exposed to temperatures higher than **40°C**.

Do not place in any **hot environments** (sauna, hot bath etc.)

Store it out of direct & indirect **sunlight** (e.g. do not leave in the car on a sunny day).

Avoid touching **hot surfaces** (fire place, heater etc.).

Your socket can be cleaned with water (up to 40°C) and mild soap or alcohol (Isopropanol).



If any irregularities and changes in the performance of the device are noted, please contact your technician.

Find amputee resources and more info on [www.amparo.world](http://www.amparo.world)



# CONFIDENCE SCHAFT

**Dein Prothesenschaft ist extrem hitzeempfindlich & sollte nicht:**

- höheren Temperaturen als **40°C** ausgesetzt werden,
- in **heißen Umgebungen** gelagert werden (Sauna, heißes Bad etc.),
- in direktem oder indirektem **Sonnenlicht** gelagert werden (z.B. an einem sonnigen Tag im Auto)
- oder **heiße Oberflächen** berühren (z.B. Kamin, Heizung).

**Dein Schaft kann mit warmem Wasser (bis zu 40°C) und milder Seife oder mit Isopropanol (Alkohol) gesäubert werden.**



Bei eventuellen Beeinträchtigungen oder Veränderungen während der Nutzung, kontaktiere deinen Orthopädiechniker.

Mehr Ressourcen und Infos unter  
[www.amparo.world](http://www.amparo.world)

