TECHNISCHE INFORMATION

DE

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TECHNICAL INFORMATION

EN

Thanks for purchasing our mini vacuum pump, please read this instruction manual carefully and thoroughly before operating the tool to get best performance.

1. TECHNICAL DATA

Functions

- Oil free, no air pollution
- Convenient for transport
- Low noise, around 50 dB

Rotationen / min (220-240V / 50 Hz) 1450 min ⁻¹	Rotationen / min (110-120V / 60 Hz) 1700 min ⁻¹	Max. Vacuum 0.8 bar	Air Volume Flow 20-23 l / min
Dimensions (LxWxH) 25,5 x 13,5 x 17 cm	Weight 3,4 kg	Power 147 W	

2. INSTRUCTIONS

1. Before connecting the vacuum pump, check that the supply voltage complies with the electrical characteristics of the motor.
2. Please follow local electrical and safety rules. Use a socket that is grounded.
3. Make sure all connections are airtight. Any leakage will affect the machine performance.
4. The vacuum pump must be used in suitable environments (well-ventilated, with an ambient temperature between +5°C and +40°C).
5. Never leave the appliance exposed to dust, acids, vapors, explosive or flammable gasses or atmospheric agents (rain, sun, fog, snow).
6. Never use the appliance bare footed or with wet hands or feet.
7. Do not allow inexperienced persons to use the vacuum pump without suitable surveillance. Keep children and animals at a distance from the work area.
8. Never allow children to touch the working machine or insert anything into the machine. It will cause the danger of burns or of electrical shock.
9. Do not open or tamper with any part of the appliance. If any problems occur, contact your sales representative.

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